REPADROP®

10 ml e

FR

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10 ml e

EN

Ophthalmic Solution for Corneal Protection - Preservative-free

1. What REPADROP® is and what is used for ?

Traumas of various types, whether mechanical, chemical, or physical, may alter the normal integrity of the corneal surface. The corneal epithelium is rich in nerve endings tasked with maintaining the corneal surface's proper morphofunctional integrity. REPADROP® is an ophthalmic solution - containing Hyaluronic Acid, Glycerilphosphorylcholine and D-panthenol - able to improve corneal neurorepair and reepithelialization. REPADROP® increases the volume of the tear film and improves its stability. The B-complex vitamins (vitamin B12 and vitamin B6), also contained in the solution, improve the protective capacity on the surface of the eye, limiting the insult of external and surgical agents. REPADROP® is a Medical Device indicated for corneal lubrication and protection and for the treatment of ocular discomfort and of pain symptoms caused by foreign body sensation, external agents, surgeries and traumas which can be improved with a local treatment. REPADROP® aids in the processes of corneal reepithelialization and neurorepair. The OSD (Ophthalmic Squeeze Dispenser) technology allows the innovative delivery of an absolutely sterile preservative free ophthalmic solution.

2. What you need to know before you use REPADROP® ?

Do not use REPADROP® if you are allergic or in case of known hypersensitivity to any component of the product (listed in Section 6).

3. How to use REPADROP® ?

Compatible with all types of contact lens.

Wash your hands before applying the product. Instill 1-2 drops in each eye 2-3 times a day or according to medical advice. Make sure that the dropper does not come into contact with the eye.

4. Warnings

- In case of adverse reaction, discontinue use and consult a doctor.
- Make sure that the bottle is intact before use.
- Do not use this product after the expiration date shown on the pack. The expiry date refers to the intact and properly stored product.
- Ocular use, do not swallow.

5. How to store REPADROP® ?

- Keep out of the reach and sight of children.
- Discard the bottle 60 days after opening.
- Close the bottle tightly after use and store at room temperature and away from heat sources.
- Do not dispose of the bottle into the environment after use.

6. Further information

Ingredients : Hyaluronic acid sodium salt, α-glycerilphosphorylcholine, vitamin B12 (cyanocobalamin), vitamin B6 (pyridoxine hydrochloride), D-panthenol, buffered isotonic solution pH 7.2.

PRESERVATIVE-FREE

Presentation : 10 ml bottle with OSD dispenser.

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