Recommended by.

VICKS

Steam Inhaler

with Vicks VapoPads®

GB STEAM INHALER
with Vicks VapoPads®

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Fig. B

Fig. C

Congratulations on your purchase of the Vicks Steam Inhaler. Your new Inhaler provides all the benefits of vapour therapy at home, at the office or when traveling.

- Soothing menthol or rosemary & lavender vapours
- Portable – use anywhere
- Simply fill with hot tap water, add scent pad and breathe
- Dishwasher safe – reusable
- 2 VapoPads® included

COMPONENTS (See page 2)

1. Top
2. Middle Piece
3. Base

DIRECTIONS FOR USE

Rinse unit out thoroughly before first use.

1. Remove top of unit from base by twisting in opposite directions and lifting off top.
2. Fill base to water line with hot (not boiling) tap water. (See Fig. A on page 3)
NOTE: If you overfill, the unit will leak.
3. Open VapoPad® packet and add to the water. Replace the top.
CAUTION: Product contains hot water. To avoid hot water spilling after filling, be sure to properly secure top housing by snapping back in place.
4. Breathe in the soothing vapours. (See Fig. B on page 3)

CARE AND CLEANING

After each use, thoroughly wash and dry your inhaler. The unit is top shelf dishwasher safe. (See Fig. C on page 3)

USE WITH VICKS VAPOPADS®

Your inhaler was designed to work with Vicks menthol or rosemary & lavender VapoPads®.

WARNING:

Pad may present a choking hazard. Keep out of reach of children and pets.

Flammable liquid and vapour (H226).

Causes skin irritation (H315).

May cause an allergic skin reaction (H317).

Causes serious eye irritation (H319).

Toxic to aquatic life with long lasting effects (H411).

Keep out of reach of children (P102).

Keep away from heat/sparks/open flames/hot surfaces. – No smoking (P210).

Avoid release to the environment (P273).

IF ON SKIN: Wash with plenty of soap and water (P302+P352).

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing (P305+P351+P338).

If skin irritation or rash occurs: Get medical advice/attention (P333+P313).

Do not use for babies weighing less than 4.5kg, for more than 16 hours in a 24 hour period and/or not more than 2 pads in a 24 hour period.

Contains: Eucalyptus oil, Cedarleaf Oil

GUARANTEE

Kaz grants 2 years guarantee commencing on the date of purchase. Any defective part will be replaced at no charge if it has not been tampered with and the appliance has been used according to our printed directions.

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© 2012, Kaz Europe SA
Vicks and other associated trademarks are owned by The Procter & Gamble Company.
Sistem Ltd.