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EN | GENTLE, RISK-FREE HYGIENE

AUDISPRAY JUNIOR - New formulation

AUDISPRAY JUNIOR contains a hypertonic seawater solution which is microbiologically controlled, combined with glycerol, which protects the skin of children's ear canals. AUDISPRAY JUNIOR avoids the use of cotton buds, which push the wax towards the eardrum.

What is AUDISPRAY JUNIOR used for?

AUDISPRAY JUNIOR softens and dissolves the wax, making it easier for it to be naturally evacuated from ear. AUDISPRAY JUNIOR contains glycerol, with hydrating and softening effects, respecting the fragility of children's ears. AUDISPRAY JUNIOR, with no propellant gas and with an anti-overpressure nozzle, permits regular, effective and risk-free hygiene.

Who can use AUDISPRAY JUNIOR?

Children aged 3 to 12 years: 1 to 2 sprays in each ear, twice a week. **Children who produce a lot of wax**, in whom AUDISPRAY JUNIOR is used every day.

How to use your AUDISPRAY JUNIOR ?

1. Point the nozzle towards the child's ear and administer 1 to 2 sprays per ear, keeping the bottle upright. In this case, please consult a doctor. It is recommended that you use AUDISPRAY JUNIOR within a year after the first use of the bottle. Do not use the product after the expiry date stated on the packaging and on the bottle.

Contraindications

Do not use in children who have grommets in their ears, are suffering from otitis or perforated eardrum, or generally in any child with a medical history (vertigo, medical or surgical treatments, etc.)

Precautions for use

If you are not sure, or if the child has pain before using the product, ask your pharmacist or doctor for advice. If a plug of earwax has already formed AUDISPRAY JUNIOR can cause the plug to swell, leading to discomfort. In this case, please consult a doctor. It is recommended that you use AUDISPRAY JUNIOR within a year after the first use of the bottle. Do not use the product after the expiry date stated on the packaging and on the bottle.

AUDISPRAY JUNIOR is a medical device which conforms to European Directive 93/42/EEC | Tel: +41 22 718 64 20 | E-mail : uk@diepharmex.com

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